

Meditec ME-SORB™

Carbon Dioxide Absorbent



Special “D” profiles of the particles provides high ratio of surface area to volume.

Low dust. Less than 0.3%

High CO₂ absorption = Low cost

Low Interaction

USP v ME-SORB™ Comparison

| USP Standards | Me-Sorb™ Properties |
|---|--|
| <ul style="list-style-type: none"> Mesh Size 4-8 (2.5mm to 5.0mm), 98% between 0.4mm and 6.3mm | <ul style="list-style-type: none"> 90% between 2.36mm and 4.8mm |
| <ul style="list-style-type: none"> Hardness of 75% (minimum) | <ul style="list-style-type: none"> 95% Typical |
| <ul style="list-style-type: none"> Dust 2% maximum | <ul style="list-style-type: none"> 0.3% Typical (0.5% maximum) |
| <ul style="list-style-type: none"> CO₂ absorption greater than 100 litres/Kg or 19% weight | <ul style="list-style-type: none"> CO₂ absorption: 140 litres/Kg or 25% weight |
| <ul style="list-style-type: none"> Moisture Content 12-19% weight (necessary to initiate the reaction with CO₂) | <ul style="list-style-type: none"> Moisture Content: 16% Typical |



Benefits

- Effective shape for CO₂ absorption
- Larger surface area relative to volume than uniformly shaped soda lime
- Increased reaction time resulting in high absorption levels
- Maximum consumption of soda lime

Low Dust

The dust content of ME-SORB™ is minimal at 0.3% (USP specification:2%). Coupled with 95% (typical) hardness ensures zero additional dust is produced during transport and handling, thus, reducing the risk of dust contamination and gas flow resistance during use.

Chemical Composition and Anaesthetic Degradation

ME-SORB™ does not contain Potassium Hydroxide (KOH). Therefore the risk of Compound A and Carbon Monoxide (CO) is minimised.


Moisture Content

ME-SORB™ is manufactured with a moisture content of 15.5%. This attribute prevents the granules from drying out during normal usage. The production of compound A and Carbon Monoxide (CO) only occurs when soda lime is very dry.


CO₂ Absorption Capacity

In low-flow anaesthesia ME-SORB™ demonstrates a highly efficient absorption capacity. Test data confirm ME-SORB™ consistently absorbs approximately 150 litres of CO₂ per kilogram of ME-SORB™ before experiencing a 0.5% CO₂ breakthrough. The ultimate benefit of this attribute is the cost saving associated to the higher absorption capacity.


ME-SORB™ and Dust

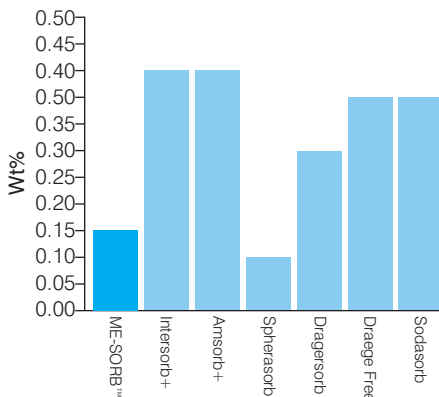

Simulated Low Flow Results

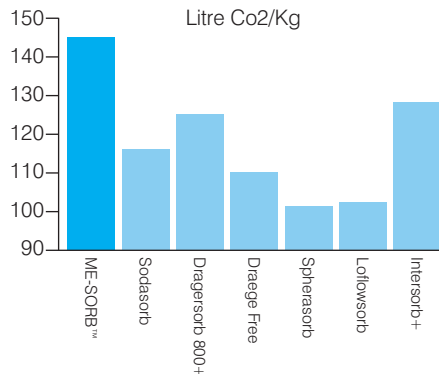

Absorption

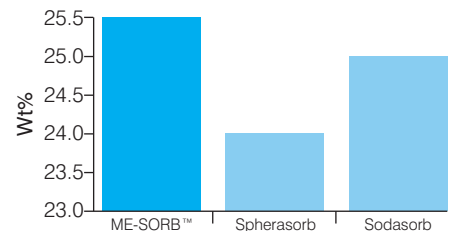

Order Information

Item Code

11001059R0 White to Violet 2 Jars (4.5Kg/5Ltr) Per Carton

11001058R0 Pink to White 2 Jars (4.5Kg/5Ltr) Per Carton



Meditec International England Limited

8, Pinner View, Harrow, Middx,
HA1 4QA, United Kingdom
Tel: +44 203 6423808

E-mail : sales@meditecengland.co.uk
Website : www.meditecengland.co.uk

Available at:

XAVIER Med Pvt. Ltd.

1-10-51/2, Road 6B, Dwarakapuram,
Dilsukhnagar, Hyderabad - 500060

www.buyxavier.com

**Customer Care: 8008895680,
Service: 8008895079**

*In view of continued product improvement Meditec International England Limited reserves the right to make changes in the equipment which may affect the information contained in this leaflet without giving any prior notice. Pictures are representative, accessories are optional and may vary at the time of supply.